



**Toilet seat Dania with cover, 50 mm raised**  
**With stabilising buffers and special hinges adapted to raised toilet seat.**

<p><b>Item number</b></p>	<p>R33000  VVS:615685000  NRF:6190584  RSK:7972415  EAN:5708590070520  HMI: 1810  NBS:N13:300</p> <p>Toilet seat, Dania, with stabilising buffers, w/cover, raised 50 mm. With raised cross bar hinge</p>		
<p><b>Colour</b></p>	<p>White</p>		
<p><b>Description</b></p>	<p>Toilet seat, Dania, with stabilising buffers, w/cover, raised 50 mm.  With raised cross bar hinge</p>		
<p><b>Materials</b></p>	<p>Seat/cover: Thermoset plastic (urea)</p> <p>Strap: Stainless steel V2A</p> <p>Raiser: Nylon (PA)</p> <p>Special buffer: ABS-plastic</p>		
<p><b>Product capacity</b></p>	<p>This product has been tested statically up to 225 kg. As the European standard for technical aids for disabled, BS/EN 12182:2012, requires a static test with safety factor 1.5, the nominal maximum load for this product is therefore 150 kg.</p>		
<p><b>Weight</b></p>	<p>3,9 kg</p>		
<p><b>Gross weight</b></p>	<p>4,2 kg</p>		
<p><b>Weight per crate</b></p>	<p>4,2 kg</p>		
<p><b>Qty per crate</b></p>	<p>1</p>		
<p><b>Crates per pallet</b></p>	<p>60</p>		





# PRESSALIT®

---

**Toilet seat Dania with cover, 50 mm raised  
With stabilising buffers and special hinges adapted to raised toilet  
seat.**

---

**Qty per pallet**

60

**Cleaning**

Use a mild soap solution to clean the toilet seat. Please take care that the seat and hinges are not left damp. For disinfecting the seat, e.g. ethanol 85% can be used. Make sure that the product is wiped down with a soft rag wrung out in clear water and dried off with a soft, dry cloth immediately after disinfection. *Avoid contact of the seat and hinges with abrasive, corrosive or chlorine-based cleaners, as these may cause damage or lead to flash rust. Therefore, when cleaning the bowl, make sure that the seat and cover are kept in an upright position until all cleaning agent has been flushed away.*

**Warranty**

Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



---

[www.pressalit.com](http://www.pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

PRESSALIT®